


## ABRIS Tests


### Pass/ Refer

**Short description:** The The Pass/Refer test is used for objective hearing evaluation of neonates/newborns. The test will check for brain stem activity in response to click stimuli at a fixed level (30,35 or 40 dB nHL). The ABRIS recording will stop if a pass criterion is met, otherwise it will use the full recording time (120 sec) and report Refer. **Work Area:** Screening **Electrode montage:** Traditional 4 electrodes montage, vertex, mastoids, and ground (cheek, low forehead). Alternative 3 electrode montage, vertex, nape, and ground (cheek, low forehead).

### Results:

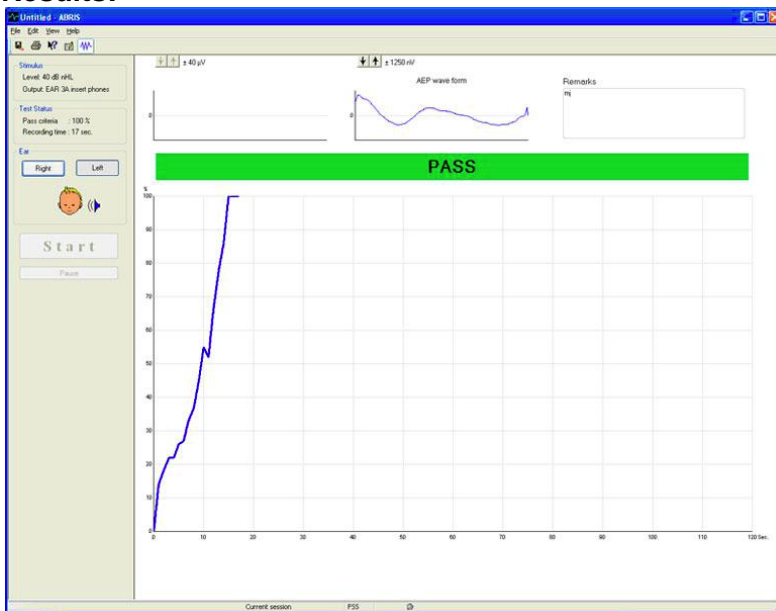

Pass test - At 40 dB nHL (30,35,40 dB nHL can be used)

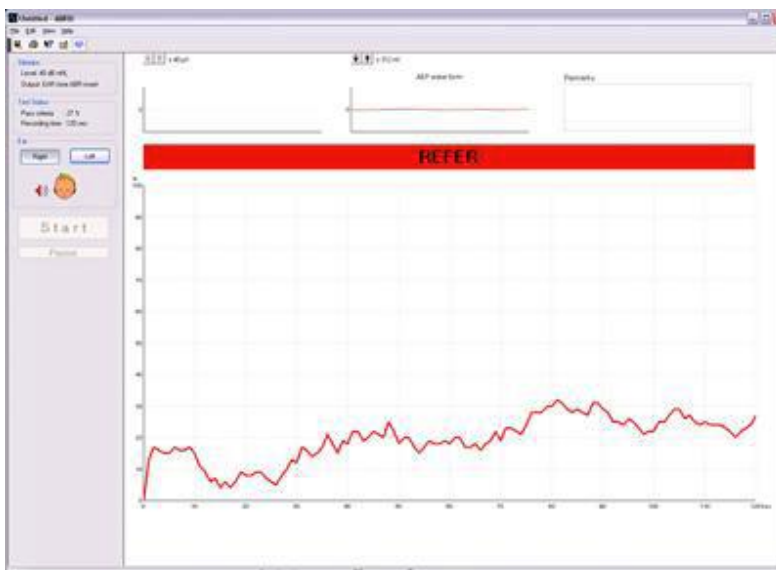

Refer test - At 40 dB nHL (30,35,40 dB nHL can be used)